FOXXMED

Single Ring Chin Strap

FMMD2062

Light

SINGLE RING

Intended Use

- ▶ FOXXMED Single Ring Chin Strap is an intuitive and easy fitting accessory which is designed to restrict the mouth from opening during Positive Airway Pressure (PAP) treatment and minimise air loss through the mouth.
- ▶ The Single Ring Chin Strap is intended for single patient re-use.
- ▶ If possible, have the Single Ring Chin Strap initially fitted by staff at your sleep disorders center or by your home care provider.

Face Circumference Measurements


- ▶ Measure around your head with a cloth ruler. The circle should go along the chin and cheek, and around the top of your head.

⚠ **NOTE:** If you experience any difficulty with the fitting of the Single Ring Chin Strap, consult your physician or home care provider.


Single Ring Chin Strap

Recommended Face Circumference: 60 - 76 cm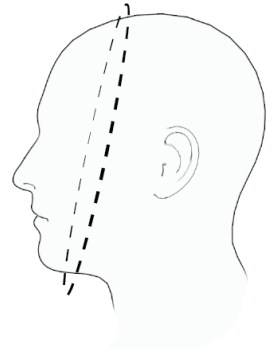

Instruction for Use

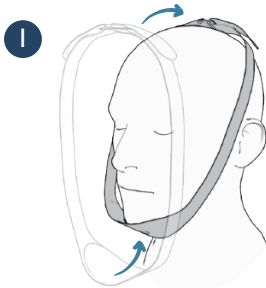Place chin support under the chin, then put another side cross over the head.

Gently adjust the Velcro Straps of both sides to secure the chin straps firmly but comfortably around your head.

Please check the tips to obtain the excellent stability for the treatment.

The chin-hole is outward.

The Velcro strap should completely hook on the other side.

The chin strap is positioned as shown in the illustration.

Warnings

- ▶ If symptoms of a dry/sore mouth/throat occur or notice any redness or irritation of your skin, discontinue use and consult your physician or sleep therapist.
- ▶ If any visible deterioration (eg. wear, fraying, excessive stretching, tears) of the Single Ring Chin Strap is apparent, the Single Ring Chin Strap should be discarded and replaced.
- ▶ Do NOT position the Single Ring Chin Strap to obstruct your mouth or restrict the airflow of the mask system.
- ▶ Do NOT overtighten or position the Single Ring Chin Strap completely beneath the chin or across your upper throat.
- ▶ Do NOT use the Single Ring Chin Strap if you have flu or cold symptoms.
- ▶ DO NOT multi-patient reuse this product. Reuse may result in compromising product integrity or cross infection.

Cleaning Instructions

- ▶ Clean the chin strap at least once each week or more often.
- ▶ Wash with laundry bag using a mild detergent. Rinse thoroughly and drip dry.

FOXXMED LTD.

No. 60, Jianjin 2nd Rd., Dongshan Township, Yilan County 269, Taiwan (R.O.C.)

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