Advanced Halo Chin Strap

FMMD2064

Comfort

ADVANCED HALO


Intended Use

- ▶ FOXXMED Advanced Halo Chin Strap is an intuitive and easy fitting accessory which is designed to restrict the mouth from opening during Positive Airway Pressure (PAP) treatment and minimise air loss through the mouth.
- ▶ The Advanced Halo Chin Strap is intended for single patient re-use.
- ▶ If possible, have the Advanced Halo Chin Strap initially fitted by staff at your sleep disorders center or by your home care provider.

Face Circumference Measurements

- ▶ Measure around your head with a cloth ruler. The circle should go along the chin and cheek, and around the top of your head.


⚠ **NOTE:** If you experience any difficulty with the fitting of the Advanced Halo Chin Strap, consult your physician or home care provider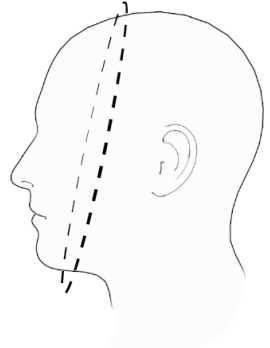


Advanced Halo Chin Strap

Recommended Face Circumference: 60 - 76 cm

Instruction for Use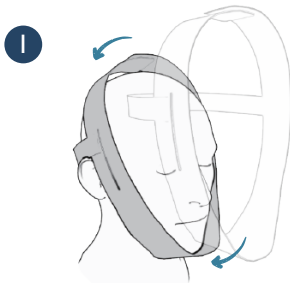

Put the back belt of chin strap onto the back of the head.

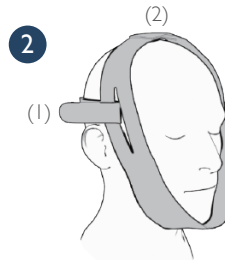Gently fasten the Velcro Straps (1) of the back to secure the chin strap for getting basic stability. Then re-adjust the straps (2) firmly but comfortably around your head.

Please check the tips to obtain the excellent stability for the treatment.

The chin support is completely cover the chin.

The Velcro strap should completely hook on the other side.

The chin strap is positioned as shown in the illustration.

Warnings

- ▶ If symptoms of a dry/sore mouth/throat occur or notice any redness or irritation of your skin, discontinue use and consult your physician or sleep therapist.
- ▶ If any visible deterioration (eg. wear, fraying, excessive stretching, tears) of the Advanced Halo Chin Strap is apparent, the Advanced Halo Chin Strap should be discarded and replaced.
- ▶ Do NOT position the Advanced Halo Chin Strap to obstruct your mouth or restrict the airflow of the mask system.
- ▶ Do NOT overtighten or position the Advanced Halo Chin Strap completely beneath the chin or across your upper throat.
- ▶ Do NOT use the Advanced Halo Chin Strap if you have flu or cold symptoms.
- ▶ DO NOT multi-patient reuse this product. Reuse may result in compromising product integrity or cross infection.

Cleaning Instructions

- ▶ Clean the chin strap at least once each week or more often.
- ▶ Wash with laundry bag using a mild detergent. Rinse thoroughly and drip dry.

FOXXMED LTD.

No. 60, Jianjin 2nd Rd., Dongshan Township, Yilan County 269, Taiwan (R.O.C.)

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